



## Specifications

Capacity:	240 ml (excluding cap)
Flow rate:	-40 kPa within 10s @ less than 2Hz pump rate
Tube diameter:	17mm
Operation:	Aspirator functions normally with automatic aspirate ejection.
Seal type:	Rolling seal diaphragm
Maintenance:	Requires annual checkups
Operating temperature:	-20 to 50°C
Dimensions (mm):	165 x 165 x 87 mm Gross 212 x 102 x 233 mm
Weight:	Gross 650g

Our policy of improvement through research and production changes may take place and may not be reflected in the specification printed in this publication. We point this out to avoid possible infringement under the Trademark Act and would ask you to check with us or your local Distributor if clarification on any point is required.

## Accessories & Spare Parts

Description
Rigid Yankauer Catheter
Carrying Case
Silicone Grease

Description
Container Seal
End Cap Seal
One-way Valve
Mesh Filter
Aspirate Container
Container Cap
Rear end Cap Assembly
Yankauer Catheter Mount (pack of 5)
Replacement 9mm Catheter (pack of 5)
Tube mount 9mm (pack of 5)

## Customer Service

There are no user serviceable parts; any repair should only be carried out by the manufacturer, the approved importer or by Service Agents specifically approved by Vitalograph - special equipment is required. For names and addresses of approved Vitalograph Distributors or Service Agents nearest to you, please contact:

### Contact us;

Steel yao.

E-mail: [steel@medinstru.com](mailto:steel@medinstru.com)

Meds Medical Equipment Co.,Ltd

**Physical Address:**  
No.1 Jiahang Road Jiaxing, Zhejiang PR.China

**Mailing Address:**  
Suite 808, Taikun International Plaza, Jiaxing, Zhejiang, PR.China

Tel: +86-57383768915  
Fax: +86-57383667955

Email: [webmaster@medinstru.com](mailto:webmaster@medinstru.com)  
Email Sales / Customer Service: [steel@medinstru.com](mailto:steel@medinstru.com)

Web: [www.medinstru.com](http://www.medinstru.com)

Printer reference: 08229 / 13

## Guarantee

### Terms of Guarantee

Subject to the conditions listed below, Vitalograph Ltd. and its associated companies, (hereinafter called the Company) guarantee to repair or at its opinion replace any component thereof, which, in the opinion of the Company is faulty or below standard as a result of inferior workmanship or materials.

The conditions of this Guarantee are:

- This Guarantee shall only apply to hardware defects which are notified to the Company or to its accredited distributor within 1 year of the date of purchase of the equipment, unless otherwise agreed in writing by the Company.
- Software (meaning computer software, or user installable modules) is guaranteed for 90 days from the date of purchase.
- The Company warrants that the software when correctly used in conjunction with the hardware will perform in the manner described in the Company's literature and user manuals. The Company undertakes to rectify at no expense to the customer any software failure notified within the period stated above, provided that the failure can be recreated and the software has been installed and used in accordance with the user manual. Notwithstanding this clause, the software is not warranted to be free of errors.
- This Guarantee does not cover any faults caused by accident, misuse, neglect, tampering with the equipment, use of consumable items or parts not approved by the Company, or any attempt at adjustment or repair other than by personnel accredited by the Company, nor does it cover reinstatement of any configuration changes caused by the installation of any software.
- If a defect occurs, please contact the supplier from whom it was purchased for advice. The Company does not authorise any person to create for it any other obligation or liability in connection with Vitalograph equipment.
- This guarantee is not transferable and no person firm or company has any authority to vary the terms or conditions of this Guarantee.
- To the maximum extent permitted by law, the Company does not accept liability for any consequential damages arising out of the use of, or inability to use any Vitalograph equipment.
- This Guarantee is offered as an additional benefit to the Consumer's statutory rights and does not affect these rights in any way.